

Lubri Spray



All-purpose Hand piece Lubricant Spray for maintenance of all dental hand pieces including high speed, slow speed and air motors.

Packing	.Product Code
600 ml Can + Nozzle	5506 / 600
150 ml Can + Nozzle	5506 / 150

Indications:

Highly effective lubrication for all high and slow-speed dental rotary handpieces. Lubri Spray contains an anti-rust agent that prevents micro rust on inner bearings. High penetrative ability to ensure thorough coating of bearing surface. Each can contain a high-volume pressure to provide maximum bearing rotation during oiling. High internal pressure of 70+Psi (4.8atm) at 20°C. Cans are compatible with the most renowned brands. Clear in colour.

Directions for use:

Insert into the back of the handpiece and spray through for one to two seconds immediately after each use and before sterilization. Wipe excess spray thoroughly.

Important:

Hold can in a vertical position when in use. Refer to handpiece maintenance instructions where applicable.

Shelf life and storage:

- Best before 24 months under proper storage
- Store at temperature between 10° C to 30° C.



Manufactured by:

Shiva Products

HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East),
Dist. Palghar, 401 208, Maharashtra, India

+919967806543 | care@shivaproducts.net

www.maarcidental.com