



## Articulating Paper [Instruction For Use]

### INTENDED USE:

Used as diagnostic device in dentistry to highlight occlusal contacts and the distribution of occlusal forces.

### COMPOSITION:

Transferable ink coated paper

### PRODUCT DESCRIPTION & PROPERTIES:

MAARC articulating paper are coated with easy transfer ink so it accurately highlights occlusal contacts. It comes in various thicknesses. MAARC Articulating Paper comes with progressive color transfer and aids in visualizing static occlusion. Because of its low thickness and good flexibility, adapts perfectly to cusps and fossae, making it extremely precise and therefore suitable for more delicate preparations.

**PRODUCT COLOR:** Red / Blue

### PROCEDURE :

- 1) Insert an articulating paper into the articulating paper holder.
- 2) Determine the direction and position of the holder so that the articulating paper is present on the side where the occlusal contact is to be verified.
- 3) Insert the holder with articulating paper into the patient's mouth so that the holder is aligned with the distal side of the last molar.
- 4) Ask the patient to bite in centric relation or to achieve maximum intercuspation.
- 5) Ask the patient to open the wide mouth and retrieve the holder with articulating paper.
- 6) The markings of the articulating paper on the occlusal surface suggest the areas of contact or intercuspation, pinpointed markings suggests a higher contact or obstruction in the occlusion.
- 7) The markings left by the articulating paper on the teeth or prosthesis can be removed by rinsing with water. Hot water, mechanical friction (toothbrush or dental floss), alcohol and isopropyl alcohol will also loosen residual color deposits.

### CAUTION / WARNING:

- 1) Ensure device to be used only by a qualified dentist or dental technician.
- 2) Paper may transfer ink easily on any surface which it comes in contact with, care should be taken to avoid contact with clothing to prevent staining.

### PRECAUTIONS:

- 1) Avoid prolonged or repeated contact with skin (allergic contact dermatitis) and avoid contact with eyes. Do not take internally.
- 2) Skin Contact: If contact with skin occurs immediately wash with soap and water on contacted part. If skin rash and sensitization or other allergic reaction occurs seek medical attention.

### CONTRAINDICATION:

It should not be used with patients who have a history of severe allergic reaction to any of the components or staining dyes.

### INTERACTIONS WITH OTHER DENTAL MATERIALS :

No such Reactions Noted . Kindly communicate and inform manufacturer in details of any instance.

### ADVERSE REACTIONS :

No such Reactions Noted . Kindly communicate and inform manufacturer in details of any instance.

### STORAGE CONDITIONS:

- 1) Keep Away from Direct Sunlight / Heat source
- 2) Keep Away from Moisture
- 3) Keep locked up and out of the reach of children.
- 4) Store at Temp. between 10°C to 40°C
- 5) Shelf life not affected after opening if stored in a cool dry place.

### DISPOSAL CONDITIONS:

Dispose in accordance with local, state and federal environmental regulations.

**This product is specially formulated for use in dentistry and to be used by qualified dental personnel.**

#### Manufactured by:

#### Shiva Products

HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East),  
Dist. Palghar, 401 208, Maharashtra, India

B-18, Dwarka Indl. Est., Naik Pada, Walsi, Vasai (East).

Dist. Palghar - 401 208, Maharashtra, India

www.maarcidental.com | care@shivaproducts.net | +919967806543

EC	REP
----	-----

Obelis S.A

Bd GénéralWahis, 53,

B-1030 Brussels, Belgium

Tel: +3227325954 | Fax: +3227326003

Email: mail@obelis.net



Revision	05
Date	06-10-2020

These instructions for use should be kept for the duration of product use.

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY"

Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER"

Symbol for "EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS"

Symbol for "TEMPERATURE LIMITATION" Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.