



Shiva Products

HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East), Dist. Palghar, 401 208, Maharashtra, India

CAD / CAM Wax Blanks

INTENDED USE :

Used for the construction of dental crown and bridgework through lost wax casting method. It can also be used for calibration of CAD / CAM Machine before milling of ceramic block in CAD / CAM Machine.

COMPOSITION :

Coloured artificial waxes.

PRODUCT DESCRIPTION & PRODUCT HANDLING:

The basic wax comes in cakes/Blanks forms. They are of standard dimensions, so will fit in respective machine holders accurately. Wax has enough hardness and toughness to serve its purpose. Wax blanks / cakes have a high melting point, hence can be manipulated easily in the lab. During use in milling machine the wax will not stick to milling tool making it easy for cleaning the milling unit.

PROCEDURE OF USE:

Wax cake/Blank is fitted in milling machine & milled as per requirement in milling machine. If required, milled wax patterns of crowns or bridges can be sent for trial fitting in patient's mouth. For the final casting, lost wax technique can be followed. During the heating cycle, temperature should be held at 250-300°C for 10-15 minutes for proper de-waxing, before increasing to Final temperature as per the requirement of the metal / investment.

CAUTION / WARNING:

- 1) Device to be used by a qualified dentist or dental technician only.
- 2) During de-waxing procedure Hot wax can cause burning of the skin – avoid contact of hot wax with skin (safety clothing if necessary). Inhalation of hot wax vapours should be avoided.
- 3) Wax should be inserted into patient mouth only after temperament and only for short durations avoid continuous contact at hot temperatures.
- 4) Avoid swallowing.
- 5) For Single Use Only

Manufactured by:

Shiva Products

HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East),
Dist. Palghar, 401 208, Maharashtra, India

B-18, Dwarka Indl. Est., Naik Pada, Waliv, Vasai (East),
Dist. Palghar - 401 208, Maharashtra, India

www.maarcidental.com | care@shivaproducts.net | +91-9967806543

EC REP

Obelis S.A

Bd GénéralWahis, 53,
B-1030 Brussels, Belgium
Tel: +3227325954 | Fax: +3227326003
Email: mail@obelis.net

CE

Revision	05
Date	14-10-2020

These instructions for use should be kept for the duration of product use.



Symbol for "BATCH CODE"



Symbol for "USE BY"



Symbol for "NON STERILE"



Symbol for "KEEP DRY"



Symbol for "CONSULT INSTRUCTION FOR USE"



Symbol for "CATALOGUE NUMBER"



Symbol for "HANDLE WITH CARE"



Symbol for "EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE"



Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"



Symbol for "KEEP OUT OF THE REACH OF CHILDREN"



Symbol for "MANUFACTURER NAME & ADDRESS"



Symbol for "TEMPERATURE LIMITATION"



Symbol for "KEEP AWAY FROM SUNLIGHT"



Symbol for "DO NOT RE-USE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.