



HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East), Dist. Palghar, 401 208, Maharashtra, India

Shiva Products

Formocresol

1. DESCRIPTION

Formocresol: formaldehyde, cresol, glycerin

2. INDICATIONS

Indicated as a disinfectant and devitalizing solution used as part of endodontic treatment.

3. CONTRAINDICATIONS

This product should not be used with individuals with known sensitivities to formaldehyde or cresol. This product should not be used with individuals who are pregnant or suspected to be pregnant without a prior physician consultation.

4. PRECAUTIONS

This product can cause severe burns of skin and mucosa. Take precautions protect skin, mucosa and eyes during use.

5. WARNINGS

Do not swallow. Do not allow material to come into contact with intact skin or mucosa. Wear eye protection when handling. Ensure proper tooth isolation, including rubber dam when applying. Keep out of reach of children. For professional dental use only.

6. DOSAGE AND ADMINISTRATION

Intracanal disinfection in permanent teeth: Isolate the tooth with a rubber dam. Clean, de-bridge and instrument the canals. Rinse and dry the canal spaces. Moisten a small cotton pellet with formocresol. Blot or wring any excess liquid from the pellet. Place the cotton pellet in the pulp chamber and seal the access hole with a temporary filling material. Recall the patient after a minimum of 48 hours and remove the pellet. Complete the endodontic treatment and restore. Formocresol pulpotomy in primary teeth: Isolate the tooth with a rubber dam. Remove visible caries. Remove the coronal pulp with an excavator or bur leaving the radicular pulp. Achieve hemostasis. Moisten small cotton pellets with formocresol. Blot or wring any excess liquid from the pellet. Place the pellets directly on the radicular pulp stumps for 5 minutes. Place a base or a core in the tooth and restore.

7. STORAGE

Store between 50°-86°F (10°-30°C). Protect from freezing.

Net Contents:

15mL

Manufactured by:

Shiva Products

HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East),
Dist. Palghar, 401 208, Maharashtra, India

B-18, Dwarka Indl. Est., Naik Pada, Waliv, Vasai (East),
Dist. Palghar - 401 208, Maharashtra, India

www.maarcidental.com | care@shivaproducts.net | +919967806543

Revision	05
Date	28-04-2021

These instructions for use should be kept for the duration of product use.

Symbol for "BATCH CODE"	Symbol for "DATE OF MANUFACTURE"	Symbol for "USE BY"	Symbol for "NON STERILE"
Symbol for "KEEP DRY"	Symbol for "CONSULT INSTRUCTION FOR USE"	Symbol for "CATALOGUE NUMBER"	
Symbol for "CAUTION"	Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"		
Symbol for "KEEP OUT OF THE REACH OF CHILDREN"	Symbol for "MANUFACTURER NAME & ADDRESS"		
Symbol for "TEMPERATURE LIMITATION"	Symbol for "KEEP AWAY FROM SUNLIGHT"		

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.