

## FIND FAST (Instructions For Use)

### INTENDED USE:

Detecting carious dentine during cavity preparation in restorative dentistry. Detection of dental caries in places difficult to identify and prevent excessive removal of normal dentin.

**COMPOSITION:** Water, Dye

### PRODUCT DESCRIPTION & PROPERTIES:

Find Fast is an important tool for the detection of decay / caries through the method of pigmentation of dental caries. Find Fast clearly differentiates clinically the two layers of carious & normal dentin by staining the outer carious dentin scarlet red yet not staining the inner normal dentin. The distinct staining of only the outer, infected carious layer provides a visible, clinically precise guide for caries removal.

### PROCEDURE:

1. Isolate carious tooth with a rubber dam prior to application of the product.
2. After washing and drying the carious tooth, place one drop of the product on a disposable brush and apply it to the cavity.
3. After 10 seconds, wash the applied surface with water, to remove the excess stain.
4. Excavate the exposed outer carious dentin which is stained red, until the hard lightly stained/non-stained dentin is seen.
5. Wash the prepared cavity with water to flush out all the carious debris from the cavity.
6. Repeat the application of FIND FAST to detect any residual carious dentin.

### NOTE:

1. In shallow open cavities Find Fast can be applied to the cavity area before opening the cavity for disclosure of undermined free enamel to facilitate opening the cavity.

2. Only the non-sensitive stainable tissue should be removed, carefully avoiding cutting the sensitive unstainable dentin tissue.

### CAUTION / WARNING:

1. Use caution to prevent the product from coming in contact with the skin or eye.
2. If the product comes in contact with skin or eye, take the following actions: Immediately wash the eye with copious amounts of water and consult a physician. On contact with skin immediately wipe it off with a cotton swab moistened with alcohol or gauze and wash with copious amounts of water.
3. Avoid direct contact with the skin and / or soft tissue to prevent hypersensitivity.
4. Do not apply the product directly from the bottle to carious cavity.

### PRECAUTIONS:

1. Do not mix the product with any other dental materials.
2. Do not use the product for any purposes other than those specified in INDICATIONS.

### STORAGE CONDITIONS:

Keep away from extreme heat or direct sunlight. The product must be stored in a cool & dry place.

**DISPOSAL CONDITIONS:** Dispose in accordance with local, state and federal environmental regulations.

**This product is specially formulated for use in dentistry and to be used by qualified dental personnel.**

**Manufactured by:**  
**Shiva Products**  
 HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East),  
 Dist. Palghar, 401 208, Maharashtra, India  
 B-18, Dwarka Indl. Est., Naik Pada, Waliv, Vasai (East),  
 Dist. Palghar - 401 208, Maharashtra, India  
 www.maarcidental.com | care@shivaproducts.net | +919967806543

Revision	01
Date	28-01-2021

These instructions for use should be kept for the duration of product use.

**LOT** Symbol for "BATCH CODE"    
 Symbol for "USE BY"    
 Symbol for "NON STERILE"

Symbol for "KEEP DRY"    
 Symbol for "CAUTION"    
 Symbol for "CONSULT INSTRUCTION FOR USE"

Symbol for "CATALOGUE NUMBER"    
 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

Symbol for "KEEP OUT OF THE REACH OF CHILDREN"    
 Symbol for "MANUFACTURER NAME & ADDRESS"

Symbol for "TEMPERATURE LIMITATION"    
 Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.