

PIT & FISSURE SEALANT

Visible Light-Cure Composite

(Instructions For Use)

INTENDED USE:

To seal the pits and fissures and very small cavities.

INSTRUCTIONS FOR USE:

- Perform prophylaxis with a non-fluoride paste oil free. Isolate and dry each tooth with air oil free.
- Apply etchant with a brush and allow etch for 60 sec.
- Rinse well and dry. If the appearance does not look "chalky" repeat the procedure.
- Choose sealant base white or clear.
- Place one drop of the pit and fissure sealant in a plastic well.
- Apply over the occlusal surface watching carefully not to trap air bubbles. Cure the pit and fissure sealant with a light from a curing lamp. After the product is cured clean the occlusal surface with a cotton pellet and check with an explorer if the cover is complete, then check the occlusion.
- Some patients can present allergy to the pit and fissure sealant light cure or any of its components. In this case suspend it use.

CAUTION / WARNING:

- Keep Away from Direct Sunlight. Keep locked up and out of the reach of children.
- Always close the cap immediately after use to prevent contamination.

CONTRAINDICATION:

The material must not be used when the techniques application described are not possible.

INTERACTIONS WITH OTHER DENTAL

MATERIALS:

- No such Reactions Noted. Kindly communicate and inform manufacturer in details of any instance.

ADVERSE REACTIONS:

No such Reactions Noted. Kindly communicate and inform manufacturer in details of any instance.

PACKAGING:

Multi dose plastic dispensing syringe, packaged with disposable applicator tips.

STORAGE:

Store in a cool and dry area (10°C to 30°C).

Shelf life (2 years from the date of manufacturing) not affected after opening if stored in a cool dry place.

DISPOSAL CONDITIONS:

Dispose in accordance with local, state and federal environmental regulations.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel only.

Manufactured by:

Shiva Products


HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East),
Dist. Palghar, 401 208, Maharashtra, India


B-18, Dwarika Indl. Est., Naik Pada, Waliv, Vasai (East),
Dist. Palghar - 401 208, Maharashtra, India


www.maarcidental.com | care@shivaproducts.net | +919967806543


Revision	00
Date	18-01-2021

These instructions for use should be kept for the duration of product use.

 Symbol for "BATCH CODE"

 Symbol for "USE BY"


 Symbol for "NON STERILE"

 Symbol for "KEEP DRY"

 Symbol for "CAUTION"

 Symbol for "CONSULT INSTRUCTION FOR USE"


 Symbol for "CATALOGUE NUMBER"

 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"

 Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"

 Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.