

Pro-Paste (Instruction For Use)

Intended Use:

1. Removal of surface, soft and hard plaque.
2. Removal of extrinsic discolouration (e.g., staining from coffee, tea or tobacco).
3. Polishing tooth surfaces during prophylactic treatment, prior to bleaching and following the removal of orthodontic devices.
4. Cleaning and polishing restorations during professional tooth cleaning.
5. Removal of residues of temporary luting material prior to definitive bonding (cementation).

Composition:

Pumice powder in aqueous base, Humectant, Thickener, Talc, Sweetener, Flavoring agent.

Product Description & properties:

- Pro-Paste is a polishing paste used for professional tooth cleaning and polishing during prophylactic treatment.
- Pro-Paste is used for polishing of teeth post dental prophylaxis to effectively clean and polish the teeth surfaces.
- Ph of Pro-paste is 7+ but below 8 to avoid demineralization of tooth surface.
- Hardness of the Pro-Paste is 6-7 on Mohs scale which makes it ideal for polishing Enamel surfaces.
- Pro-Paste is suitable for use in all patients without any age or gender restrictions.
- Pro-paste is available in mint flavor to enhance patient acceptability.

Procedure:

- Remove the cap of the container and dispense some of the material onto a dappen dish.
- A rubber cup or soft bristle brush may be used on tooth surfaces for polishing / cleaning.
- Apply a small amount of paste on the tooth surface to be polished / cleaned, also fill the rubber cup with paste for best results.
- In order to avoid potential heat generation and splashing, appropriate rotational speed of 2,000 to 3,000 rpm should be used when working with the paste.
- Use the rubber cup on each tooth surface for approximately 10 seconds.
- Make Sure that the edge of the cup goes below the free gingival margin into the gingival sulcus.
- Continue procedure until all exposed surfaces of the teeth have been thoroughly cleaned and polished.

- After the application, rinse the cleaned surfaces thoroughly with water, remove remaining material by means of suction and ask the patient to rinse out their mouth thoroughly.
- Dental floss can be used to remove excess paste from interproximal areas.

Caution / Warning:

- Avoid prolonged or repeated contact with skin, mucosa and eyes.
- Avoid inhalation of fumes.
- Do not consume or take internally.
- If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water.
- If contact with eyes occurs immediately flush with running water and get medical attention immediately.
- Consult a physician in case of any hypersensitivity reaction.
- Contact with the skin may cause irritation or hypersensitivity.

Precautions:

- All tooth surfaces should be thoroughly scaled or prophylaxis (scaling) should be performed before polishing.
- Do not apply excess pressure while polishing the tooth surface as it may damage teeth & tooth structure.

Contraindications: It should not be used with patients with a history of allergic reaction to any of the components.

Interactions With Other Dental Materials: No such Reactions Noted. Kindly communicate and inform the manufacturer in details of any instance.

Adverse Reactions: No such Reactions Noted. Kindly communicate and inform the manufacturer in details of any instance.

Storage conditions:

- 1) Keep Away from Direct Sunlight and any source of heat.
- 2) Keep it out of the reach of children.
- 3) Store at Temp. between 10°C to 30°C.
- 4) Store in a cool & dry place.
- 5) Keep the cap of the container closed when not in use to prevent drying of the paste.

Disposal Conditions: Dispose in accordance with local, state and federal environmental regulations.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel.

Manufactured by:
Shiva Products

HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East),
Dist. Palghar, 401 208, Maharashtra, India

B-18, Dwarika Indl. Est., Naik Pada, Waki, Vasai (East),
Dist. Palghar - 401 208, Maharashtra, India

@ www.maarcidental.com | care@shivaproducts.net
☎ +919967806543

Revision	01
Date	05-05-2021

These instructions for use should be kept for the duration of product use.

- Symbol for "BATCH CODE"
 Symbol for "DATE OF MANUFACTURE"
 Symbol for "USE BY"
 Symbol for "NON STERILE"
- Symbol for "KEEP DRY"
 Symbol for "CONSULT INSTRUCTION FOR USE"
 Symbol for "CATALOGUE NUMBER"
- Symbol for "CAUTION"
 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
- Symbol for "KEEP OUT OF THE REACH OF CHILDREN"
 Symbol for "MANUFACTURER NAME & ADDRESS"
- Symbol for "TEMPERATURE LIMITATION"
 Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.