



# Shiva Products

HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East), Dist. Palghar, 401 208, Maharashtra, India

## SOFT - LINER

### INTENDED USE:

- 1) For the temporary re-basing (relining) of acrylic dentures.
- 2) Conditioning Tissue.

### COMPOSITION:

Polymer and monomer of EMA

### PRODUCT DESCRIPTION:

Soft denture relining material

### PROCEDURE & PRODUCT HANDLING:

#### 1. PREPARATION

Clean the contact surface thoroughly using pumice and water and abrade the area to be relined.

#### 2. MIXING

The standard powder/liquid ratio is 2.2g/1.8g.

#### Note:

Consistency may be varied by changing powder/liquid ratio.  
Measure into the glass jar and mix for 30-60 seconds.

#### 3. RELINING

- a) Coat the mixture onto the denture as evenly as possible.
- b) Insert the denture in the mouth, muscle trim and retain it in the centric occlusal position for 4-5 minutes.
- c) Remove and inspect for defects.
- d) Dip the denture into cold water, and trim away excess at the periphery with a heated sharp instrument.

### CAUTION / WARNING:

1. Device to be used by a qualified dentist or dental technician only.

### PRECAUTIONS:

1. Avoid contact of the device with skin and mucosa other than oral tissues, avoid contact with eyes. Avoid inhalation of fumes. Do not take internally.
2. Eye contact: if the product comes in contact with eye, it may cause irritation. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
3. Skin Contact: Avoid contact with skin, it may cause skin sensitization. If contact with skin occurs immediately wipe off thoroughly with cotton and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention.
4. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

### CONTRAINDICATIONS:

1. Do not use the product if the patient is allergic to any of the ingredients. Interactions With Other Dental Materials: No such Reactions Noted. Kindly communicate and inform manufacturer in details of any instance.

### ADVERSE REACTIONS:

No such Reactions Noted. Kindly communicate and inform manufacturer in details of any instance.

### PACKAGING:

Multi dose Containers containing 50g Powder & 50ml Liquid.

### STORAGE CONDITIONS:

- Keep Away from Direct Sunlight. Keep locked up and out of the reach of children.
- Store at a temperature between 4°C to 25°C.
- Shelf life (2 years from the date of manufacturing) not affected after opening if stored in a cool dry place.
- Always close the cap immediately after use

### DISPOSAL CONDITIONS:

Dispose in accordance with local, state and federal environmental regulations.

#### Manufactured by:

#### Shiva Products

HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East),  
Dist. Palghar, 401 208, Maharashtra, India

B-18, Dwarka Indl. Est., Naik Pada, Waliv, Vasai (East),  
Dist. Palghar - 401 208, Maharashtra, India

www.maarcidental.com | care@shivaproducts.net | ☎ +919967806543

Revision	06
Date	9-11-2020

These instructions for use should be kept for the duration of product use.



Symbol for "BATCH CODE"



Symbol for "USE BY"



Symbol for "NON STERILE"



Symbol for "KEEP DRY"



Symbol for "CONSULT INSTRUCTION FOR USE"



Symbol for "CATALOGUE NUMBER"



Symbol for "CAUTION"



Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"



Symbol for "KEEP OUT OF THE REACH OF CHILDREN"



Symbol for "MANUFACTURER NAME & ADDRESS"



Symbol for "TEMPERATURE LIMITATION"



Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.