



## Tray Adhesive (Instructions For Use)

**INTENDED USE:** Tray Adhesive is indicated for application to impression trays to enhance the adhesion of impression materials to intraoral impression trays including perforated and non-perforated stock metal and plastic, custom acrylic trays.

**COMPOSITION:** Adhesive compatible with impression materials

### PRODUCT DESCRIPTION & PROPERTIES:

MAARC tray adhesive is used to enhance the adhesion of impression materials to stock metal or plastic (solid or perforated) or custom acrylic impression trays. Certain impression materials may get separated from the impression tray upon removal from the mouth leading to distortion errors. Using the Maarc tray adhesive helps in securing the impression firmly to the tray upon removal from the mouth.

### PROCEDURE:

- Select and/or prepare a suitable tray. Prior to application of Maarc Tray Adhesive, trial fit tray for proper fit and extension.
- After proper fit is assured, rinse and thoroughly dry the tray.
- Brush a thin layer of Maarc Tray Adhesive onto the tray surfaces that will be covered with impression material and allow to air dry for at least three minutes, but not more than 10 minutes.
- Repeat application if more than 10 minutes have elapsed after application.
- Avoid placement of excess adhesive onto tray surfaces. Retention is not enhanced and may be compromised by placement of a thicker layer of tray adhesive onto tray surfaces.
- Mix and place desired impression material directly into the coated impression tray as per selected impression material directions.

### CAUTION / WARNING:

- Do not breathe the vapors.
- Use only in well ventilated areas.
- Avoid prolonged inhalation. Avoid prolonged or repeated contact with skin and eyes. Irritation and possible corneal damage may result.
- Skin rash or other allergic reactions may result in susceptible individuals.
- Do not take internally.
- Ensure device to be used only by a qualified dentist or dental technician.

### PRECAUTIONS:

Skin Contact: If contact with skin occurs immediately wash with soap and water on contacted part. If skin rash and sensitization or other allergic reaction occurs seek medical attention.

**CONTRAINDICATION:** It should not be used with patients who have a history of severe allergic reaction to any of the adhesive components.

**INTERACTIONS WITH OTHER DENTAL MATERIALS:** No such Reactions Noted . Kindly communicate and inform the manufacturer in details of any instance.

**ADVERSE REACTIONS:** No such Reactions Noted . Kindly communicate and inform the manufacturer in details of any instance.

### STORAGE CONDITIONS:

- Keep Away from Direct Sunlight / Heat source
- Keep Away from Moisture
- Keep locked up and out of the reach of children.
- Store at Temp. between 5°C to 25°C
- Shelf life not affected after opening if stored in a cool dry place,

**DISPOSAL CONDITIONS:** Dispose in accordance with local, state and federal environmental regulations.

**This product is specially formulated for use in dentistry and to be used by qualified dental personnel.**

Manufactured by:

**Shiva Products**

HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East), Dist. Palghar, 401 208, Maharashtra, India

■ B-18, Dwarka Indl. Est., Naik Pada, Waliv, Vasai (East), Dist. Palghar - 401 208, Maharashtra, India

☎ www.maarcidental.com | ✉ care@shivaproducts.net | ☎ +919967806543

| Revision | 01         |
|----------|------------|
| Date     | 03-11-2020 |

**These instructions for use should be kept for the duration of product use.**

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY"

Symbol for "CAUTION" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER"

Symbol for "DO NOT USE IF PACKAGE IS DAMAGE" Symbol for "KEEP OUT OF THE REACH OF CHILDREN"

Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "TEMPERATURE LIMITATION"

Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.