



PLAQUE-D (Instructions For Use)

Intended Use:

Dental plaque disclosing gel.

Indications:

To identify plaque before oral prophylaxis. Patient education and motivation about oral hygiene.

Composition:

Sodium Saccharin, Gelant, Staining agent, Water.

Product Description & Properties:

Plaque-D is a water based gel contains staining agent. It helps in differentiating plaque by colour and aids in patient oral hygiene education. The gel is easy to apply and provides good control over the flow.

Procedure:

1. Apply Plaque-D onto the tooth surfaces using a suitable instrument (microbrush, swab or tooth brush).
2. After 10 seconds, Instruct the patient to lightly rinse their mouth with water.
3. Identify the plaque by observing the color of the gel or stained plaque. A pink colour on the tooth surface indicates plaque accumulation.
4. Educate the patient about the accumulated plaque using a mirror or intra-oral camera. Instruct the patient with proper tooth cleaning or oral hygiene practices and remove any remaining plaque and disclosing gel.
5. Ask the patient to rinse mouth thoroughly and spit the residues. This product is not intended to be ingested.

Caution / Warning:

1. Do Not use the product if the patient is allergic to any of the constituents of the device.

2. Use caution to prevent the product from coming in contact with the skin or eye.
3. If the product comes in contact with skin or eye, take the following actions: Immediately wash the eye with copious amounts of water and consult a physician. On contact with skin immediately wipe it off with a cotton swab moistened with alcohol or gauze and wash with copious amounts of water and soap.
4. Avoid direct contact with the skin and / or soft tissue to prevent hypersensitivity.

Precautions:

1. Do not mix the product with any other dental materials.
2. Do not use the product for any purposes other than those specified in INDICATIONS.
3. Do not use on patients having erosive gingivitis, mouth ulcers, any oral injury or any such conditions.

Storage Conditions:

Keep away from extreme heat or direct sunlight. The product must be stored in a cool & dry place.

Disposal Conditions:

Dispose in accordance with local, state and federal environmental regulations.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel.

Manufactured by:
Shiva Products
 HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East),
 Dist. Palghar, 401 208, Maharashtra, India
 B-18, Dwarka Indl. Est., Naik Pada, Waliv, Vasai (East),
 Dist. Palghar - 401 208, Maharashtra, India
 www.maarcidental.com | care@shivaproducts.net | +919967806543

Revision	01
Date	27-08-2021

These instructions for use should be kept for the duration of product use.

- Symbol for "BATCH CODE"
 Symbol for "DATE OF MANUFACTURE"
 Symbol for "USE BY"
 Symbol for "NON STERILE"
- Symbol for "KEEP DRY"
 Symbol for "CONSULT INSTRUCTION FOR USE"
 Symbol for "CATALOGUE NUMBER"
- Symbol for "CAUTION"
 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
- Symbol for "KEEP OUT OF THE REACH OF CHILDREN"
 Symbol for "MANUFACTURER NAME & ADDRESS"
- Symbol for "TEMPERATURE LIMITATION"
 Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.