

Zirprophy

Benefits of Zirprophy

- 1) Minimum sensitivity as minimal abrasion
- 2) Very effective stain removing
- 3) Polishing of teeth

Indication

- 1) Zirprophy paste is indicated for use to effectively clean and polish teeth.

Composition

Zirconium silicate with flavoring humectants

Contra-indications

Zirprophy should not be used in patients with a known allergy to any of the components

Warnings

As mild abrasive in nature can cause eye irritation upon contact. If contact with eyes occur, immediately flush eyes for 10-15 mints. If irritation persists seek medical attention

Precautions

- 1) All tooth surface should be thoroughly scaled
- 2) Wear protective glasses and gloves
- 3) Physical & radiographs examination should be done prior of application
- 4) Don't over pressurize as it may damage teeth & teeth structure

Instruction of use

Dispense required quantity in container/dappen dish

Deposit small amount of paste on tooth to be processed, also fill rubber cup with paste for best results. Fresh paste is provided for each surface.

Run rubber cup at SLOW SPEED over each tooth surface for approximately 10 seconds.

Make Sure that the edge of the cup goes below the free margin of the gingival into the gingival sulcus. Continue procedure until all exposed surfaces of the teeth have been thoroughly cleaned and polished. A rubber cup of soft bristle brush may be used on occlusal surfaces.

Dental floss can be used to remove excess paste through interproximal areas.

Shiva Products

Mfg. site: B18, Dwarka Ind. Estate, Naik Pada, Valiv, Vasai (E), Thane - 401 208. Maharashtra. INDIA.

Website: www.maarcidental.com,

Email ID: care@shivaproducts.net,

Customer Care: +919967806543

Revision	04
Date	07-10-2020

These instructions for use should be kept for the duration of product use.



Symbol for "BATCH CODE"



Symbol for "USE BY"



Symbol for "NON STERILE"



Symbol for "KEEP DRY"



Symbol for "CONSULT INSTRUCTION FOR USE"



Symbol for "CATALOGUE NUMBER"



Symbol for "CAUTION"



Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"



Symbol for "KEEP OUT OF THE REACH OF CHILDREN"



Symbol for "MANUFACTURER NAME & ADDRESS"



Symbol for "TEMPERATURE LIMITATION"



Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.