



GSOL (Instructions For Use)

Please read carefully before use

INTENDED USE:

GSOL is intended for use during repeated root canal treatments to remove gutta-percha cones from the root canal.

INDICATIONS:

During re-root canal treatment of teeth.

COMPOSITION:

Orange oil (>95%)

PRODUCT DESCRIPTION & PROPERTIES:

It is an orange oil (D-limonene) based Gutta Percha Solvent used for removing gutta percha at the time of Re-root canal treatment.

METHODS OF USE:

1. Isolate the teeth using a rubber dam. Prepare the access cavity and apply a few drops of GSOL using a sterile syringe in the root canal from which GP Point is desired to be removed.
2. Wait for 2 to 3 minutes for the GP points to soften, use Gates Glidden drills or Peeso Reamers for removal of the softened GP points.
3. If the GP is dry or crumbly, re-apply GSOL and remove the GP with the selected root canal instrument.
4. In curved canal hand instruments such as H-files can be used for removing the softened GP points.
5. Care to be taken not to extrude the GSOL beyond apex.

CAUTION/ WARNING:

- Do not use it beyond the apex of the canal.
- Avoid direct exposure to sunlight.
- Keep the container closed all the time when not in use.
- Store in a cool and dry area. (10°C to 30°C). Improper storage can cause change in properties and may affect the efficiency of GSOL.
- Once dispensed from the container, do not re-use the device.

PRECAUTIONS:

- Avoid contact with skin, mucosa and eyes.
- Avoid inhalation of fumes.
- Do not consume or take internally.
- Contact with the skin/mucosa may cause irritation or hypersensitivity.

- If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water.
- If contact with eyes occurs immediately flush with running water and get medical attention immediately.
- Consult a physician in case of any hypersensitivity reaction.

CONTRAINDICATIONS:

1. Device should not be used with patients with a history of allergic reaction to any of the components.
2. Device should not be used beyond the confinements of the root canal and teeth.

INTERACTIONS WITH OTHER DENTAL MATERIALS:

No such Reactions Noted. Kindly communicate and inform the manufacturer in details of any instance.

ADVERSE REACTIONS:

No such Reactions Noted. Kindly communicate and inform the manufacturer in details of any instance.

PACKAGING:

Multi dose Amber Glass bottle.

STORAGE CONDITIONS:

Store in a cool and dry environment. Always close the cap immediately after use to prevent contamination.

SHELF LIFE:

2 years from the date of manufacturing.

DISPOSAL CONDITIONS:

Dispose in accordance with local, state and federal environmental regulations.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel only.

Manufactured by:

Shiva Products

HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East), Dist. Palghar, 401 208, Maharashtra, India

B-18, Dwarka Indl. Est., Naik Pada, Waliv, Vasai (East), Dist. Palghar - 401 208, Maharashtra, India

www.maarcidental.com | care@shivaproducts.net | t +919967806543

EC REP

Obelis S.A

Bd GénéralWahis, 53,

B-1030 Brussels, Belgium

Tel: +3227325954 | Fax: +3227326003

Email: mail@obelis.net

CE
0123

Revision	06
Date	11-09-2021

These instructions for use should be kept for the duration of product use.

Symbol for "BATCH CODE" Symbol for "DATE OF MANUFACTURE" Symbol for "USE BY" Symbol for "NON STERILE"

Symbol for "KEEP DRY" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER"

Symbol for "EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CAUTION"

Symbol for "TEMPERATURE LIMITATION" Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.