

Green Tracing Sticks



Description: Green tracing sticks is a thermoplastic material supplied in the form of sticks (type-1 low fusing impression compound). It is a low fusing material primarily used for border extensions on custom impression trays as it attains uniform plasticity. Attains high viscosity at comfortable temperatures 55-65°C making it ideal for recording the full depth of the sulcus, recording of the borders during impression making for preparation of removable dental prosthesis, taking functional and relining impressions. Provides wide working area due to extra working time & quick hardening property when immersed in cold water.

Thickness: 113 x 9 mm dia weighing appx. 10g each

Packing	Product Code
10 Sticks	1501/010
15 Sticks	1501/015
100 Sticks	1501/100

Intended use / Indications: Recording border extensions on custom impression trays. Making of functional and relining Impressions

Properties & Advantages:

- Suitable for muscle trimming work.
- Fast and uniform plasticity.
- Quick hardening in cold water.
- Wide working area.

Shelf life and storage:

- Store at Temp between 10°C to 30°C.
- Best before 60 months under proper storage.



Manufactured by:
Shiva Products

HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East),
Dist. Palghar, 401 208, Maharashtra, India

☎ +919967806543 | ✉ care@shivaproducts.net

🌐 www.maarc dental.com