

D-Vitz (Instructions For Use)

D-Vitz paste, Arsenic free. Paste for pulp devitalization

INGREDIENTS:

Paraformaldehyde, Eugenol, Lidocaine, Blue Color

PROPERTIES :

D-vitz should be used when a preparation for pulp devitalization is advisable and when an arsenic-free preparation is preferred. The active ingredient is Paraformaldehyde, a coagulant which acts on album and confers the required fibrous consistency to the pulp to facilitate its subsequent extirpation.

INDICATIONS :

Devitalization of the pulp.

DIRECTIONS FOR USE :

- Eliminate, as far as possible, the dentine of the decayed cavity in order to ensure close contact with the pulp. Close contact of the paste with the pulp accelerates the devitalizing process.
- Insert a small quantity of paste into the base of the cavity and seal hermetically with a temporary obturation.
- Devitalization is completed within about 7 days.
- If the preparation cannot be brought in close contact with the pulp, a second treatment might be necessary in a 2nd seat.
- After devitalization, a permanent root canal filling can be placed once the pulp is totally removed and pain relief is complete.

WARNING :

Contains paraformaldehyde which can be toxic by inhalation or ingestion. Contact with the skin may cause irritation, burns or hypersensitivity. When using this product, gloves and protective glasses should be worn. In case of contact with the skin, wash with plenty of soap and water. If skin irritation occurs, get medical advice / attention. In case of contact with the eye, rinse cautiously with soft, clean water for 15 minutes, holding the eyelids open and call a doctor immediately. In case of swallowing, rinse the mouth with water, drink a lot of water and consult a doctor. Keep the person exposed at rest, do not force vomiting. Do not use in cases of known hypersensitivities to any of these ingredients.

STORAGE :

Store at room temperature between 10°C and 30°C. close the container carefully after use.














SHELF LIFE : 2 Years

PACKAGING : Jar of 10gms / Multi dose plastic dispensing syringe, with pre-bent applicator tips/cannulas.

For Dental Professional use only.

 Manufactured by: Shiva Products HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East), Dist. Palghar, 401 208, Maharashtra, India  B-18, Dwarka Indl. Est., Naik Pada, Waliv, Vasai (East), Dist. Palghar - 401 208, Maharashtra, India  www.maarcidental.com care@shivaproducts.net +919967806543	<table border="1"> <tr> <td>Revision</td> <td>06</td> </tr> <tr> <td>Date</td> <td>03-09-2021</td> </tr> </table>	Revision	06	Date	03-09-2021
Revision	06				
Date	03-09-2021				

These instructions for use should be kept for the duration of product use.

-  Symbol for "BATCH CODE"
  Symbol for "DATE OF MANUFACTURE"
  Symbol for "USE BY"
  Symbol for "NON STERILE"
-  Symbol for "KEEP DRY"
  Symbol for "CONSULT INSTRUCTION FOR USE"
  Symbol for "CATALOGUE NUMBER"
-  Symbol for "CAUTION"
  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
-  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"
  Symbol for "MANUFACTURER NAME & ADDRESS"
-  Symbol for "TEMPERATURE LIMITATION"
  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.