

Relief Wax (Instructions For Use)

INTENDED USE:

Ortho relief wax- to relieve the sharper edges of orthodontic brackets from pricking.

COMPOSITION:

Silicone Rubber raw

PRODUCT DESCRIPTION:

Translucent in color and does not appear as an opaque wax coating on the surface of brackets.

PROPERTIES:

Soft and easy to manipulate, for easy application on the surface of orthodontic brackets. The unique properties of Ortho wax make it perfect for protecting your gums from the metal edges of braces. The wax is solid at room temperature but can be softened and shaped by the warmth of your hands.

PROCEDURE & PRODUCT HANDLING:

- Pinch off a piece of wax about the size of a small pea
- Squeeze until it softens and roll it into a ball, then flatten the ball slightly
- Gently Insert the relief wax on the surface of the orthodontic brackets
- Apply slight finger pressure to push the wax onto the brackets, adapting the wax onto the surface of the bracket

CAUTION / WARNING:

1. Device to be used by a qualified dentist or dental technician only.

PRECAUTIONS:

1. Avoid prolonged or repeated contact with skin (allergic contact dermatitis) and avoid contact with eyes. Do not take internally.

2. Skin Contact: If contact with skin occurs immediately wash with soap and water on contacted part. If skin rash and sensitization or other allergic reaction occurs seek medical attention.

CONTRAINDICATIONS:

It should not be used with patients who have a history of severe allergic reaction to any of the components of the device.

INTERACTIONS WITH OTHER DENTAL MATERIALS:

No such Reactions Noted. Kindly communicate and inform the manufacturer in details of any instance.

ADVERSE REACTIONS:

No such Reactions have been Noted. Kindly communicate and inform manufacture in details of any instance.

STORAGE CONDITIONS:

1. Keep Away from Direct Sunlight.
2. Keep locked up and out of the reach of children.
3. Store at a temperature between 10°C to 30°C.

SHELF LIFE: (2 years from the date of manufacturing) not affected after opening if stored in a cool dry place.

DISPOSAL CONDITIONS: Dispose in accordance with local, state and federal environmental regulations.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel.

Kindly retain the instructions for use & packaging box until the complete use of the product / expiry.

Manufactured by:

Shiva Products

HO: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East), Dist, Palghar, 401 208, Maharashtra, India

B-18, Dwarka Indl. Est., Naik Pada, Waliv, Vasai (East), Dist, Palghar - 401 208, Maharashtra, India

www.maarcidental.com | care@shivaproducts.net | T: +919967806543

EC REP








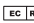







Obelis S.A

Bd Général Wahis, 53,
B-1030 Brussels, Belgium
Tel: +3227325954 | Fax: +3227326003
Email: mail@obelis.net

CE

Revision	01/00
Date	30.07.2022

Kindly retain the instructions for use & packaging box until the complete use of the product / expiry.

-  Symbol for "BATCH CODE"
  Symbol for "COUNTRY OF MANUFACTURE"
  Symbol for "USE BY"
  Symbol for "NON STERILE"
-  Symbol for "MEDICAL DEVICE"
  Symbol for "CONSULT ELECTRONIC INSTRUCTION FOR USE"
  Symbol for "CATALOGUE NUMBER"
-  Symbol for "EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE"
  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
-  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"
  Symbol for "MANUFACTURER NAME & ADDRESS"
  Symbol for "CAUTION"
-  Symbol for "KEEP DRY"
  Symbol for "TEMPERATURE LIMITATION"
  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.